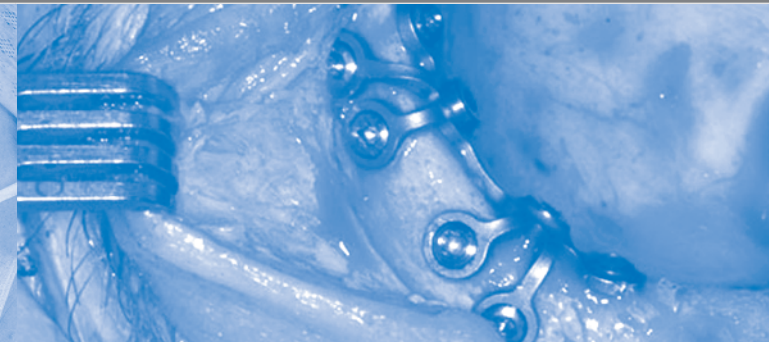
Implants

Epiplating

Titan-Plattensystem
Titanium plate system
Sistema de placas de titanio
Système de plaques en titane
Sistema di placche in titanio

Modelländerungen vorbehalten
Patterns are subject to change without further notice
Salvo modificaciones
Tous droits réservés des changements de modèle
Ci riserviamo la facoltà di cambiamenti nei modelli

Für Druckfehler übernehmen wir keinerlei Haftung
We are not liable for misprints
Declinamos toda responsabilidad por los errores de impresión que pueda contener este documento
Nous déclinons toute responsabilité en cas d'erreurs d'impression
Non ci assumiamo nessuna responsabilità per eventuali errori di stampa

Gedruckt in Deutschland
Printed in Germany
Impreso en Alemania
Imprimé en Allemagne
Stampato in Germania
Trykket i Tyskland

© Copyright 09/2012, MEDICON eG, Tuttlingen, Germany

Ti-Epiplating System

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**Titanium plate system for osseo-integrated bone
anchorage of craniofacial prosthesis and bone
conduction hearing systems**

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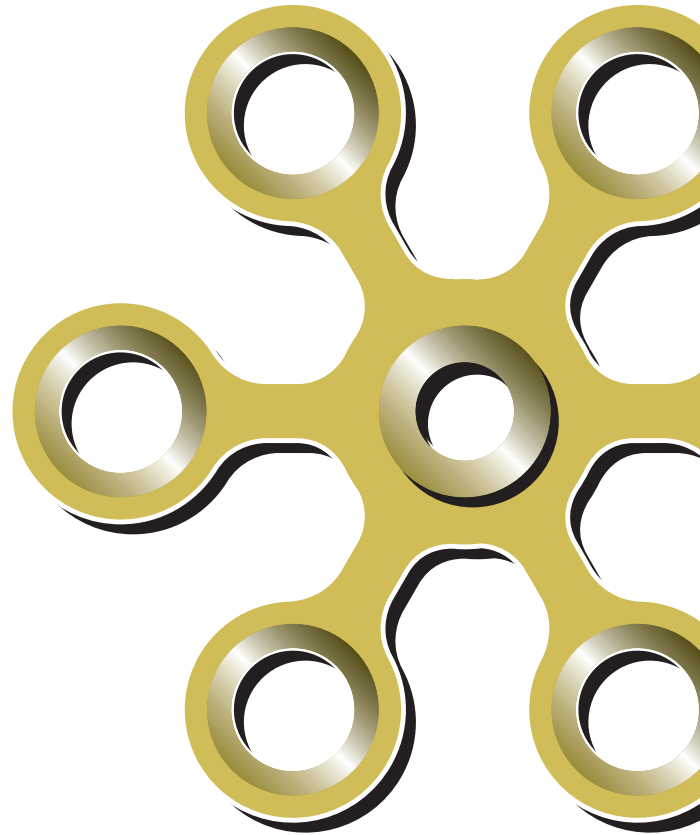
Rehabilitation of patients with craniofacial defects caused by innate malformation, trauma or tumour surgery, was revolutionized by the method of direct bone anchorage of craniofacial prosthesis (epithesis) and bone anchored hearing systems. By no other means, such a good retention of the prosthesis could be achieved. Thereby, the physician is able to offer a reliable and cosmetically sophisticated concept for rehabilitation after surgical tumour therapy. This is especially important, when surgery means an ablative procedure with complete resection of the nose, the ear or the orbita. Osseo-integrated bone anchorage by titanium implants and the following prosthetic care is of great benefit for the patients with craniofacial defects.

The **MEDICON "Ti-Epiplating System"** spreads the forces acting on the superstructure at first onto the plates, thereafter onto the multiple bone screws. This eliminates the inconvenience of solitary implant systems, where all forces impact lies on the fixture. Furthermore, it represents a special advantage in patients with scarce or low quality bony bed, as it is often the case in the orbital and nasal region, but also in the very well pneumatized mastoid process. The attachments are established remote from the site of the osseointegrated bone screws, i.e. the skin penetration of the attachments is not near to the bone screws.

The **"Ti-Epiplating System"** implants are made of biocompatible titanium and the design of the individual plates is adjusted to the anatomy of the different parts of the skull.

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Titanium hearing aid plate for fixing Ponto, the partially implanted, bone anchored hearing system by Oticon Medical.

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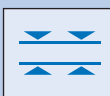
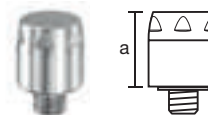
Technovent Limited
 Unit 5
 Kingsway Buildings
 Bridgend Ind. Est.
 Bridgend
 South Wales, UK
 CF31 3YH
 Telefon: +44 (0) 1656 768566
 Telefax: +44 (0) 1633 896588
 www.technovent.com

Verpackungseinheit "X" Stück
 Packing unit "X" pieces
 Paquete de "X" unidades
 Emballage de "X" pièces
 Confezione da "X" pezzi

Titan Epiplating System für die Bügelkonstruktion
Titanium Epiplating System for bar construction
Sistema Epiplating de titanio para la construcción de percha
Système Epiplating en titane pour le montage avec barre
Sistema Epiplating in titanio per la costruzione dell'archetto auricolare

Schraubendrehergriff
 Screwdriver handle
 Mango para atornillador
 Manche de tournevis
 Manico per cacciavite

10 cm - 4"

75.23.78A

Schraubendreherklinge
 Screwdriver blade
 Hoja de atornillador
 Lame de tournevis
 Lama per cacciavite

7 cm - 2¾"

75.23.69

Schraube für Goldhülse, Titan
 Screw for gold sleeve, titanium
 Tornillo para vaina dorada, titanio
 Vis pour douille en or, titan
 Vite per bussola d'orata, titanio

68.80.65

Goldhülse
 Gold sleeve
 Vaina dorada
 Douille en or
 Bussola d'orata

68.80.64

Für die epithetische Abformung
 For prostetic impression procedure
 Para el modelaje epitético
 Pour le modelage prothétique
 Per la modellazione epitetica

Abformpfosten
 Impression cap
 Poste de impresión
 Pilier de moulage
 Perno per modellazione

68.80.60

Modellpfosten
 Abutment replica
 Modelo de poste
 Pilier modèle
 Perno per modello

68.80.62

The background is a blue-tinted photograph of a surgical site. A metal surgical instrument is visible at the top. The central focus is a surgical site with several circular titanium implants. A white graphic overlay, consisting of interconnected circles and lines, is positioned over the top left and center of the image. The text 'Ti-Epiplating System' is centered in the lower half of the image.

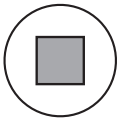
Ti-Epiplating System


Titanschrauben, selbstschneidend
Titanium screws, self-tapping
Tornillos de titanio, autorascontes
Vis en titane, autotaraudeuses
Viti in titanio, autofilettanti

“EASY GRIP”

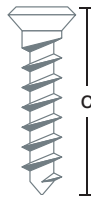
Schraubenkopf mit konischem Innenvierkant
 Screw head with tapered inside square
 Tornillo de cabeza con cuadrado interior cónico
 Tête de vis carré intérieur conique
 Testa della vite con quadrangolo interno conico

c

	Self-tapping Ø 2,0 mm	Emergency screw Ø 2,3 mm
4,0 mm	68.75.54A	
5,5 mm	68.75.25A	68.75.42A
7,0 mm	68.75.27A	68.75.43A

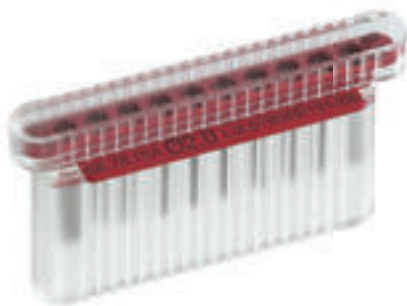Verpackungseinheit 10 Schrauben im Magazin (nicht steril).
 Packing unit 10 screws in storage tray (non-sterile).
 Paquete de 10 tornillos en cartucho (no estéril).
 Emballage unitaire de 10 vis en cartouche (non stérile).
 Confezione da 10 viti nella cartuccia (non sterile).

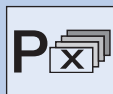
Schraubendrehergriff und Schraubendreherklinge
 Screwdriver handle and screwdriver blade
 Mango y hoja para atornillador
 Manche et lame de tournevis
 Manico e lama per cacciavite

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
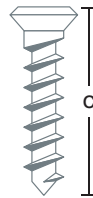
c	Self-tapping Ø 2,0 mm	Emergency screw Ø 2,3 mm
4,0 mm	68.94.00A	
5,5 mm	68.94.01A	68.94.09A
7,0 mm	68.94.02A	68.94.10A

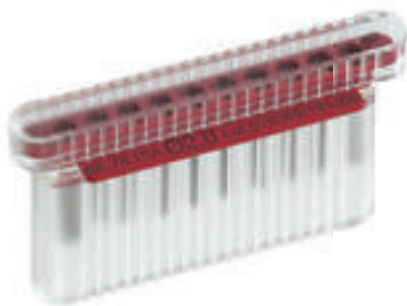Verpackungseinheit 10 Schrauben im Magazin (nicht steril).
 Packing unit 10 screws in storage tray (non-sterile).
 Paquete de 10 tornillos en cartucho (no estéril).
 Emballage unitaire de 10 vis en cartouche (non stériles).
 Confezione da 10 viti nella cartuccia (non sterile).

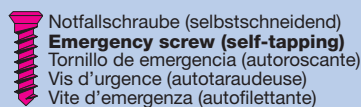
Schraubendrehergriff und Schraubendreherklinge
 Screwdriver handle and screwdriver blade
 Mango y hoja para atornillador
 Manche et lame de tournevis
 Manico e lama per cacciavite

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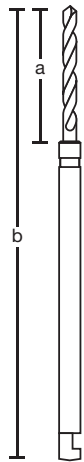
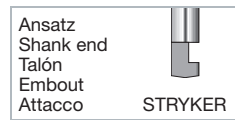
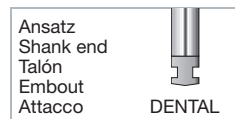
Implant tray (empty) with sliding lid for storage and sterilization of plates, screws, burrs, applicators and accessories

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Teflon tray (empty) with sliding lid for storage and sterilization of titanium magnetic inserts

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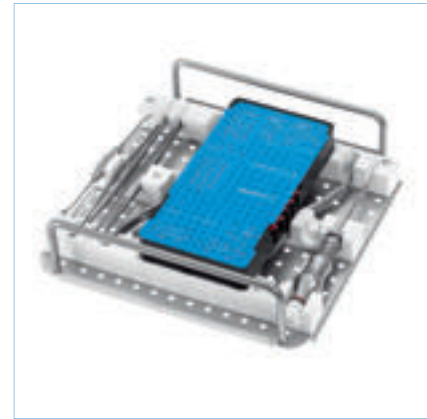
Tray (empty) with handles for storage and sterilization of: 3 plate bending forceps (68.76.41, 390.55.30); 1 plate and 1 screwholding forceps (68.76.53, 68.76.54); 1 cutting plier (14.30.13); 1 screwdriver handle (75.23.98A); 1 replaceable screwdriver blade (75.23.00, 75.23.01), 1 applying instrument (68.76.43); 1 ratchet with screwing insert (69.00.16 + 75.23.64) and two implant cassettes (68.80.78 + 68.80.76)

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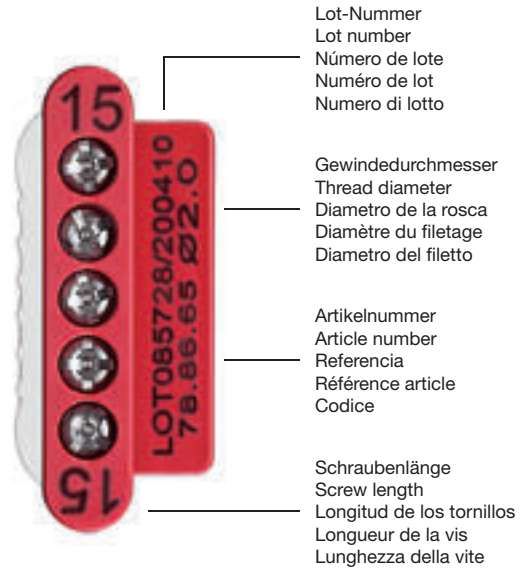
The "TraceID"-pins as well as the four identification specifications on the screw cartridges guarantee a reliable traceability of implants and screws.

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Aluminium sterilization container, blue, anodized, incl. paper filter, perforated lid, bottom non-perforated, for tray 68.80.77 (page 16)

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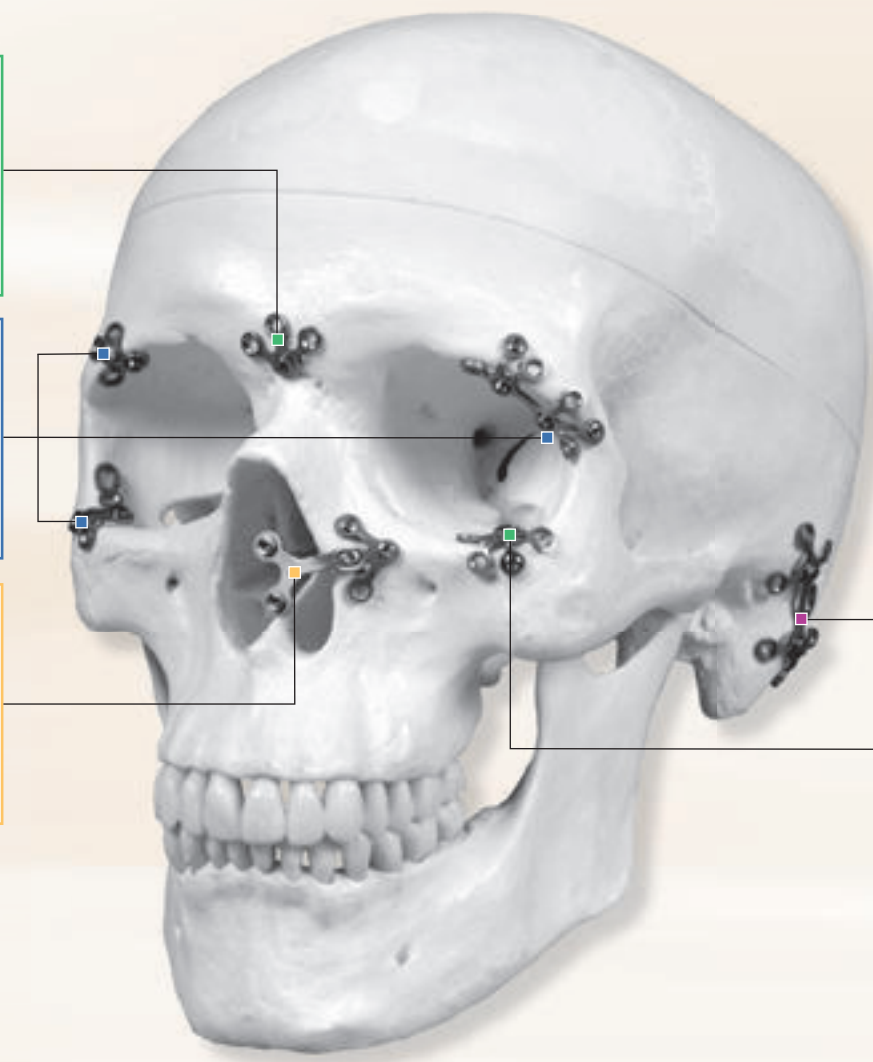
The suggestions mentioned hereunder represent procedures of magnetic or bar construction epiplating, resulting from a long lasting experience of our cooperating partners. These suggestions may be amended individually according to personal experience or anatomical conditions.

Please choose the items needed from the corresponding discipline.

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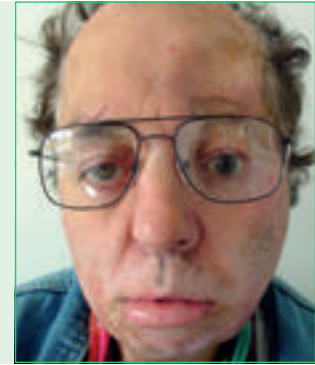
Possible implant position of the bridge plate (68.80.35). The bridge plate is designed to bridge larger defects and can be used in the entire facial area.

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Surgical procedure for bone anchoring of the prosthetic plate (68.80.35). Covering screws (68.80.41) are fastened.

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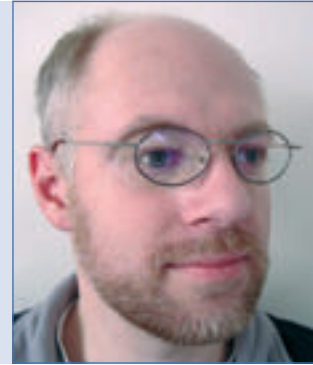
Preferred implant positions of orbital plate (68.80.30) and universal plate (68.80.34).

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Surgical procedure for bone anchoring of the prosthetic plate (68.80.30) for the orbita (top) and the universal plate (68.80.34) (bottom). Covering screws (68.80.41) are fastened.

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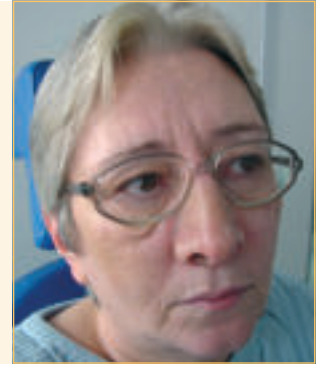
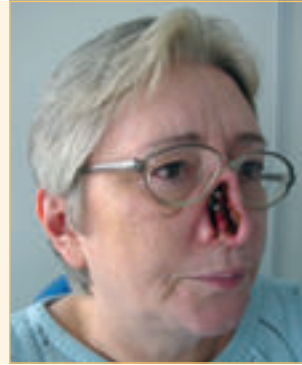
Preferred implant positions of the nose plate (68.80.31). The lateral position should be preferred over the position on the glabella.

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Surgical procedure for bone anchoring of the prosthetic plate (68.80.31) for the nose. Covering screws (68.80.41) are fastened.

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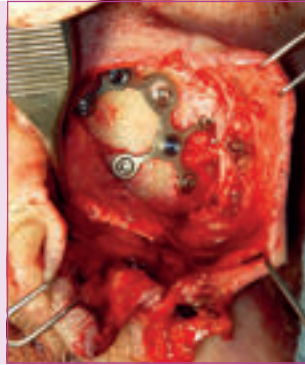
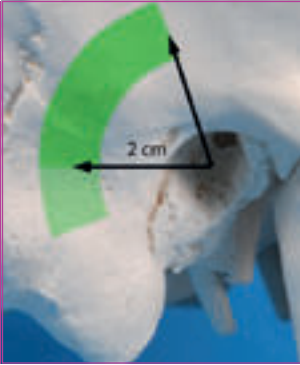
Preferred implant positions of ear plates (68.80.32) and (68.80.40).

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Surgical procedure for bone anchoring of the prosthetic plate (68.80.32) for the ear. Covering screws (68.80.41) are fastened.

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Healed implant with bar construction.

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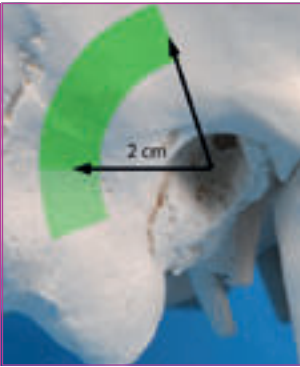
Surgical procedure for bone anchoring of the prosthetic plate (68.80.40) for the ear. Covering screws (68.80.41) are fastened.

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Preferred implant positions of the hearing aid plate (68.80.52).

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Surgical procedure for bone anchoring of the hearing aid plate (68.80.52). Covering screws (68.80.41) are fastened.

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Screwed-on abutment for bone anchored hearing aid.

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Special Notes:

The atraumatic surgical technique common in implantology is absolutely essential for avoiding heat trauma. Sharp spiral drills, water irrigation and reduced drilling speed are particularly important. For anaplastology measures it has to be ensured that no pressure is exerted on the skin covering the implant plate. As a guideline at least three bone screws should be inserted into the magnet or basic post.

In principle the surgery may be performed as a **one-stage** or **two-stage** procedure. A stress-free healing period of about 6 weeks is necessary. Usually the one-stage procedure is preferred.

For children, patients with radiotherapy or patients with unfavorable bone conditions a two-stage procedure should be performed with a stress-free and lengthened healing period of 3 to 6 months.

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
Before bending the plate the threads are secured by cover screws. The cover screws (68.80.41) are placed with a square-head applicator (68.80.42).

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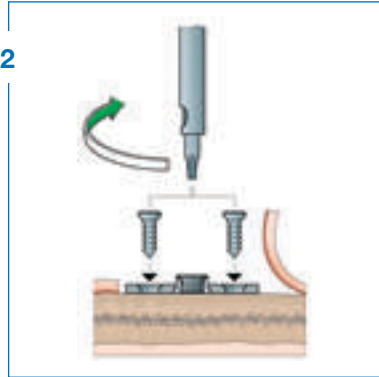
After skin incision, the periosteum is elevated. The plate is adapted to the bony contour and fixed with bone screws depending on bone thickness. Bone screws are available in different lengths of 4.0/5.5/7.0 mm. In the one-stage procedure, the skin flap may be thinned out to the elevation of the basic post at this stage.

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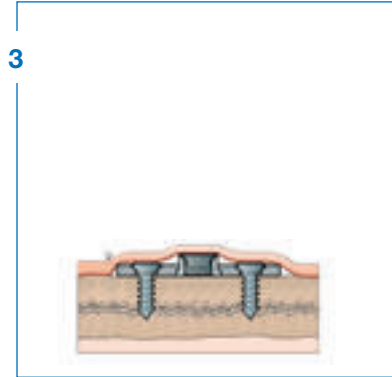Two-stage procedure:

The second operating stage is executed after a healing period of 6 to 12 weeks. After re-opening of the wound, the skin is now incised and thinned out to the elevation of the basic post as needed. Periosteum and skin are closed again. The procedure will be performed in the same way as follows:

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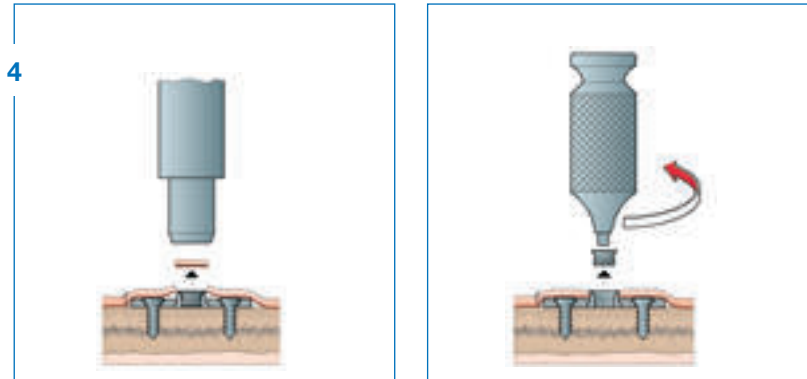
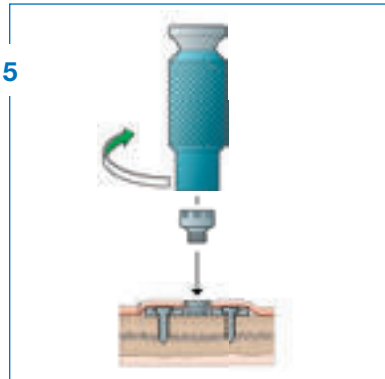
After skin closure a hole is punched over the thread (standard disposable biopsy punch, Ø 4.0 mm). The cover screw is removed with the square-head applicator.

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
Depending on skin-thickness the basic posts (68.80.45, 68.80.55, 68.80.56) are screwed in with the octagonal applicator (68.80.46).

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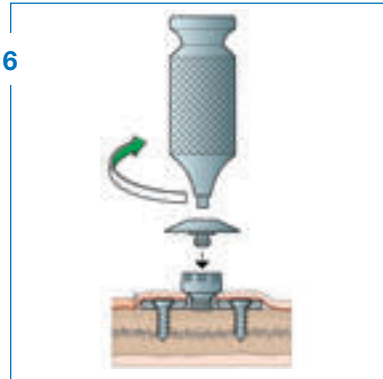
A healing cap (68.80.48) can be screwed onto the basic post with a square-head applicator. The healing cap (68.80.49) is placed when a magnet is screwed on immediately.

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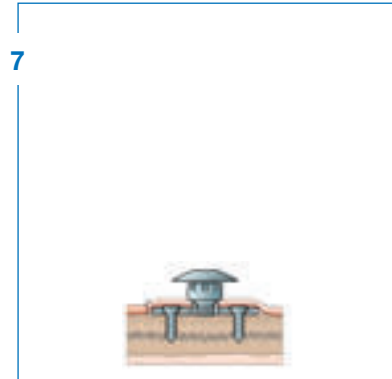
Situation with screwed on basic post together with a healing cap.

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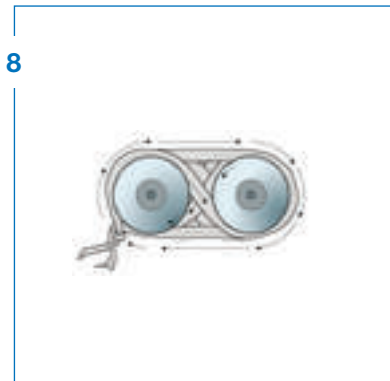
A compression bandage is applied as shown in the diagram.

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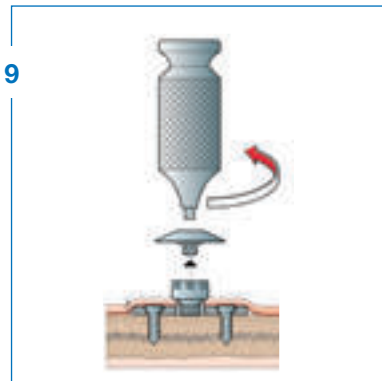
After positive healing period the compression bandage and the healing caps are removed.

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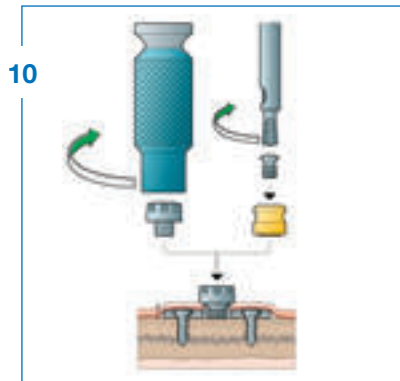Either a magnet (68.80.68/69) or a gold sleeve (68.80.64) for soldering is screwed onto the basic post.

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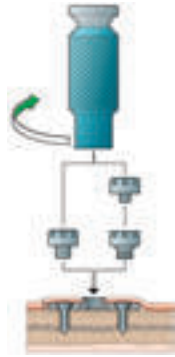
Magnet application:

Both the basic post and the magnets can be screwed into position using the octagonal applicator. The magnet (68.80.68) can be screwed directly into the plate. The thread of the magnet (68.80.69) is smaller and can only be used with the basic post.

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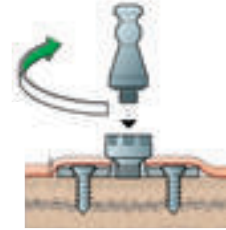
The magnet impression cap (steco) is inserted in order to define the implant position.

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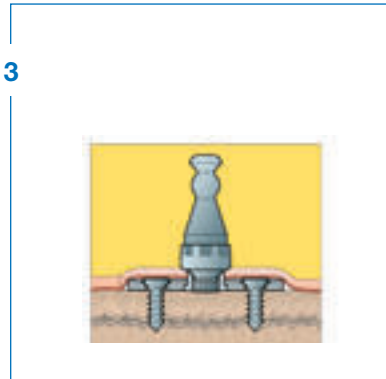
The impression cap (68.80.60) is placed manually for bar construction.

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
The impression material (e.g. silicone) is then applied to the defect area via the impression cap (68.80.60 or steco).

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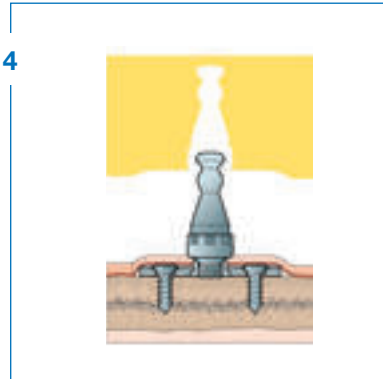
The silicone impression material is then removed from the defect after hardening ...

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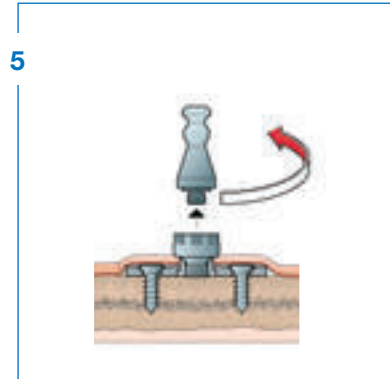... and the impression cap (68.80.60) must be unscrewed. In case the **magnet** is applied, the silicone impression material is removed together with the magnet impression cap (steco).

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
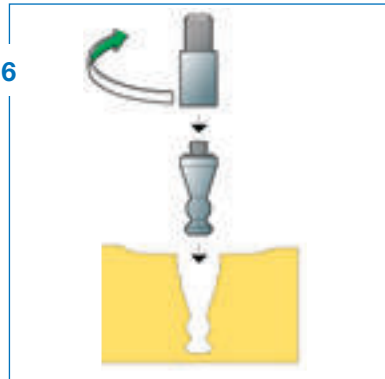
The model implant (68.80.62 or steco) is then screwed on or positioned on the impression cap (68.80.60 or steco) ...

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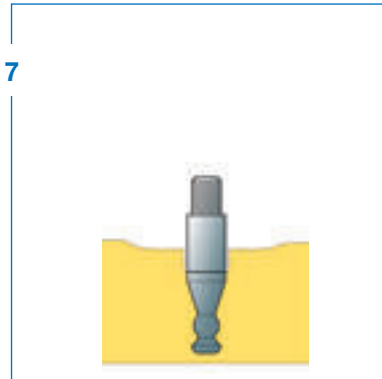
... and repositioned into the silicone impression material.

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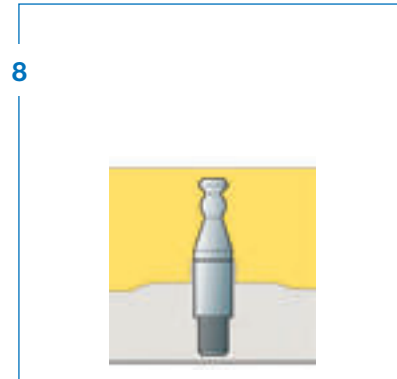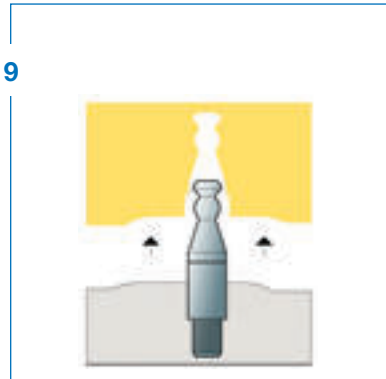
The impression material with the repositioned model implant is then casted with the model plaster.

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
Once the plaster model has hardened, the silicone impression material is removed to define the position of the defect area and the support elements.

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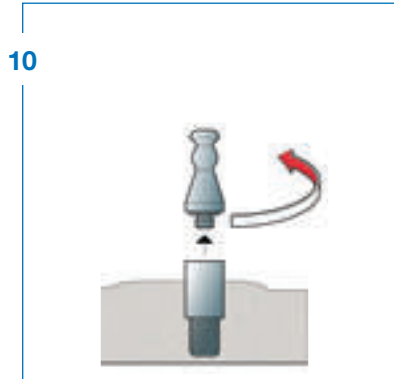
The impression cap (68.80.60 or steco) is then unscrewed once again or removed. The model implant (68.80.62 or steco) remains in the plaster model.

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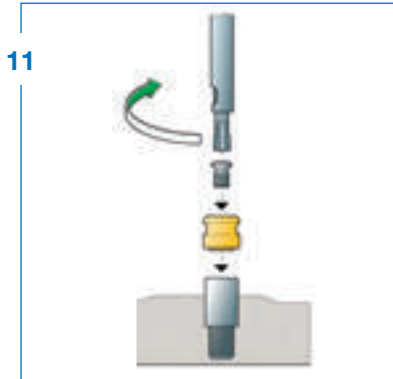For soldering the bar construction the gold sleeve (68.80.64) is attached with the fastening-screw (68.80.65) to the model implant (68.80.62) using the slot pin screw driver (75.23.78A + 75.23.69) or, if a **magnet** is used, the holding magnets are tested and checked.

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The subsequent procedure to insert the prosthesis demands competence and experience on the part of the anaplastologist.

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Federspil PA (2009) Ear epistheses as an alternative to autogenous reconstruction. Facial Plast Surg 25: S. 190-203

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Federspil PA. Implant-retained craniofacial prostheses for facial defects. In: Pau HW, ed. Technology for quality of life – implants and biomaterials in otorhinolaryngology. Mönchengladbach: Rheinware Verlag; 2011. (Current topics in otorhinolaryngology – head and neck surgery; Vol. 8). p. 259-85.

www.medicon.de

e-mail: sales@medicon.de

homepage: www.medicon.de

Medicon eG

Gänsäcker 15
D-78532 Tuttlingen

P.O. Box 44 55
D-78509 Tuttlingen

Tel.: +49 (0) 74 62 / 20 09-0
Fax: +49 (0) 74 62 / 20 09-50

e-mail: sales@medicon.de
www.medicon.de

Germany

made in Germany

Gedruckt in Deutschland
Printed in Germany
Impreso en Alemania
Imprimé en Allemagne
Stampato in R.F.G.
Trykkes i Tyskland

Modelländerungen vorbehalten
Patterns are subject to change
Salvo modificaciones
Tous droits réservés des changements de modèle
Ci riserviamo la facoltà di cambiamenti nei modelli

© Copyright 9/2012, MEDICON eG, Tuttlingen

451.09.21